

# OLSON MASK PATTERN

Created by: Clayton Skousen & Rose Hedges


VERSION 1 || 03/23/20  
FOR INTERNAL USE

## Materials Needed



- Cotton Weave Fabric (~ $\frac{1}{2}$  yard is enough to make 2-3 masks)
- All-Purpose Thread
- Hair Ties (2)\*
- Scissors
- Doubled-Sided Skin Adhesive\*

\* Adding hair ties is optional. If they are put on at the hospital, this will allow for different sizes to be made. Double-sided tap will be applied at the hospital.

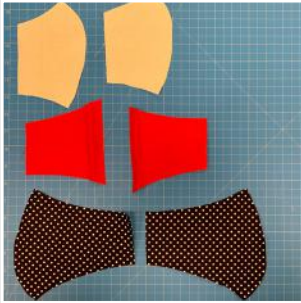

1. Using the patterns, cut one of each shape. You will need six (6) total pieces.



2. Sew face 1 and face 2 together along side 3.



3. Sew mouth 1 and mouth 2 together along side 3.



4. On mouth 1 and mouth 2 sew fold side 5 over  $\frac{1}{4}$  inch and sew down.



5. On cheek 1 and cheek 2 fold side 6 over  $\frac{1}{4}$  inch and sew down.



6. Match cheek 1 and mouth 1 at the dotted line (7). Sew a 1-inch tack at side 1 and side 2 where the two pieces overlap.

7. Repeat Step 6 for cheek 2 and mouth 2.



8. With the fronts of the fabric facing each other, sew along the entire perimeter of the mask.



9. Use one of the slits between the mouth and the cheek to turn the mask right-side out.

DO NOT do step 10 or 11 if making this for a health facility.



10. Add a hair tie to each side by folding the fabric on the end of the mask over the tie and sew down.



11. Apply double-sided skin tape around the contour of the mask to create the required seal.

### How to launder and inspect the Olson Mask with or without elastic attached:

Wash and inspect prior to use.

1. Wash with machine water setting Hot.

Masks should be washed after each use or when they become moist or soiled. Use is defined as one shift per healthcare employee. It is acceptable to wash masks at home, because they are not being used for direct patient care or as PPE.

2. Tumble dry.
3. After each washing:
  - a. Inspect seams of mask. Rule out rips, tears, and stains.
  - b. If elastic is in place, stretch elastic to ensure it does not tear or break when stretched.
  - c. If rips, tears, or stains are identified, discard mask.

## ADDING FILTER



### Adding the filter to the Olson Mask:

(Filters can be found in many products such as vacuum bags, furnace filters, etc.)

1. Use the pattern template (on the last page) to cut out the size of the filter from your source.
2. Fold the cut filter in half and insert into the pocket.
3. Open the folded filter inside of the pocket, ensuring that it reaches up to the bridge of the nose and down to the chin.


Pattern | MOUTH & CHEEK

1" x 1"  
Test Square

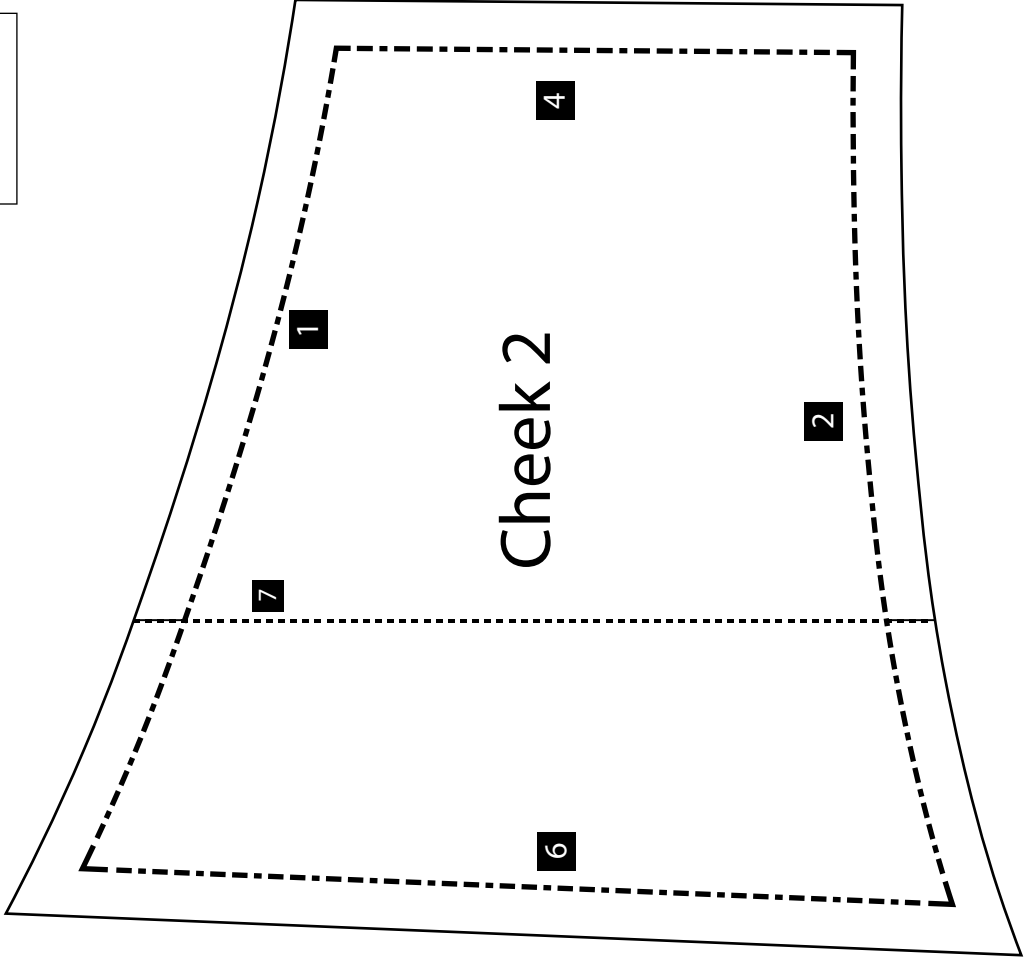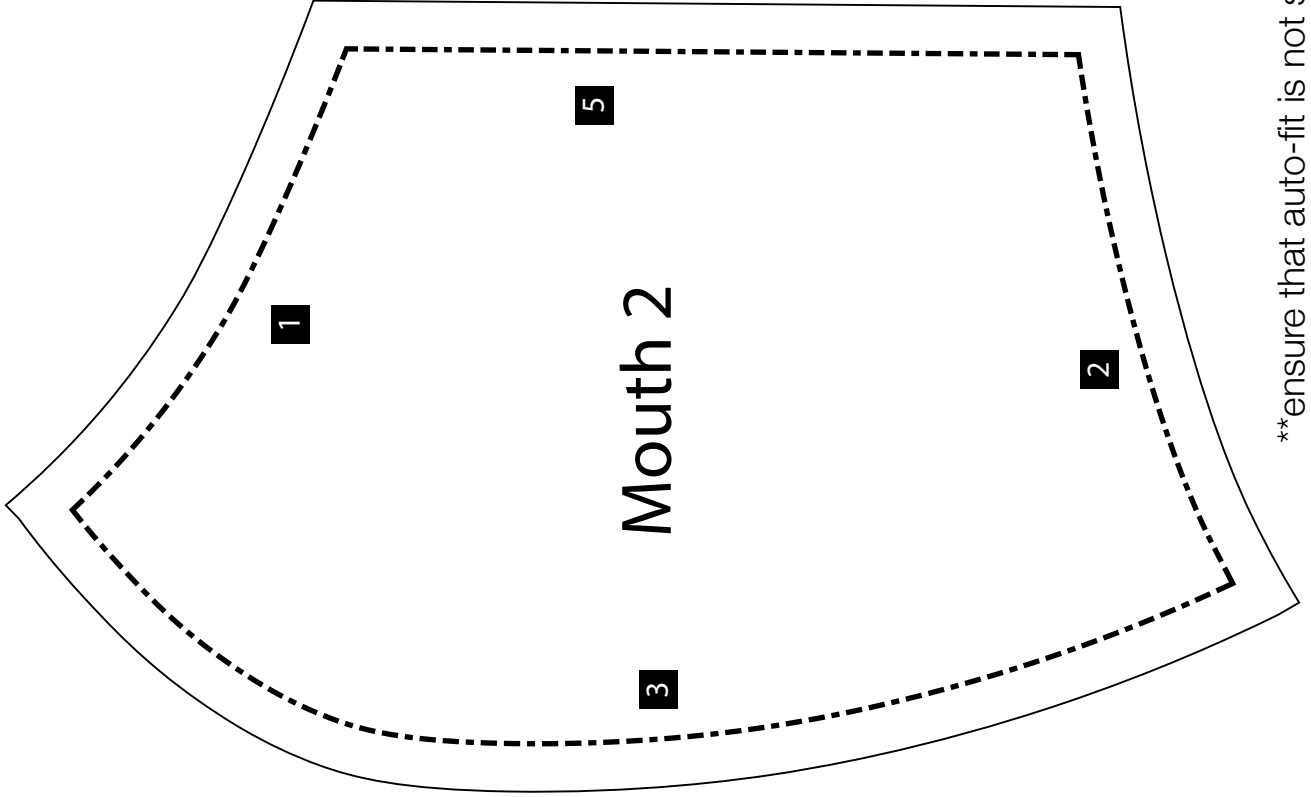

\*\*ensure that auto-fit is not selected when printing to ensure pattern measurements are not altered

Pattern | FACE

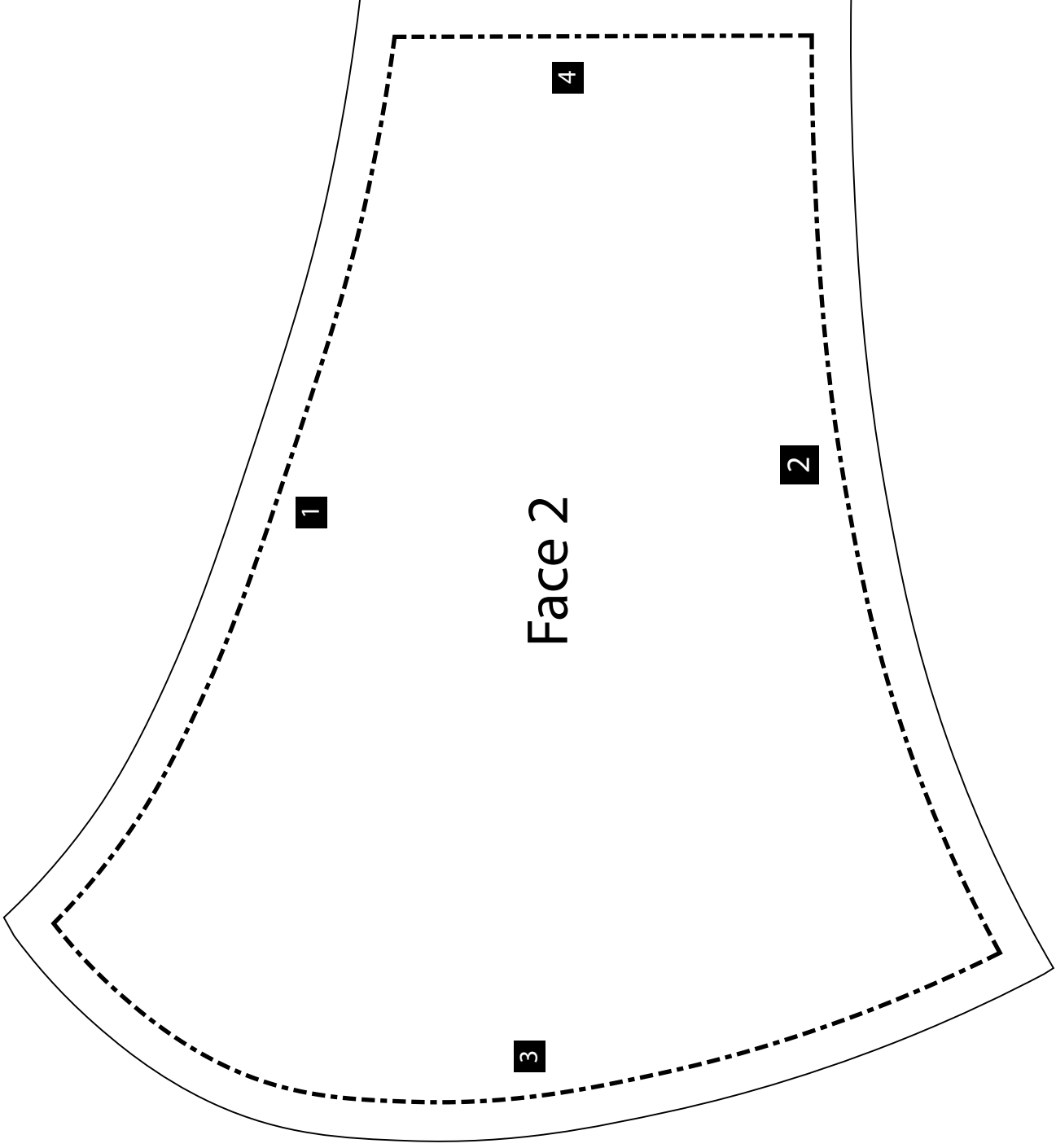


1" X 1"

Test Square

\*\*ensure that auto-fit is not selected when printing to ensure pattern measurements are not altered


1" x 1"  
Test Square

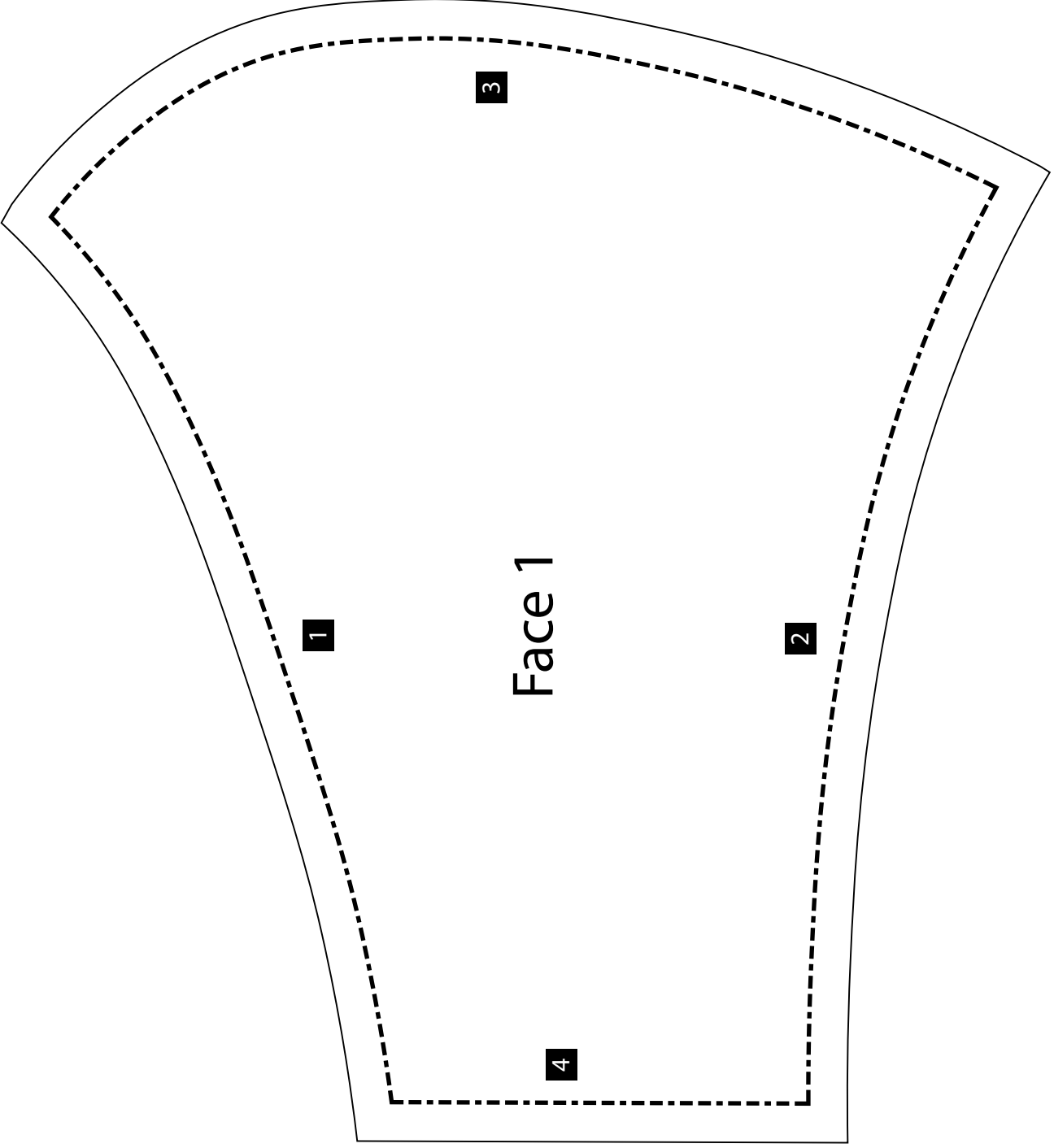

\*\*ensure that auto-fit is not selected when printing to ensure pattern measurements are not altered


Pattern | MOUTH & CHEEK

1" x 1"  
Test Square

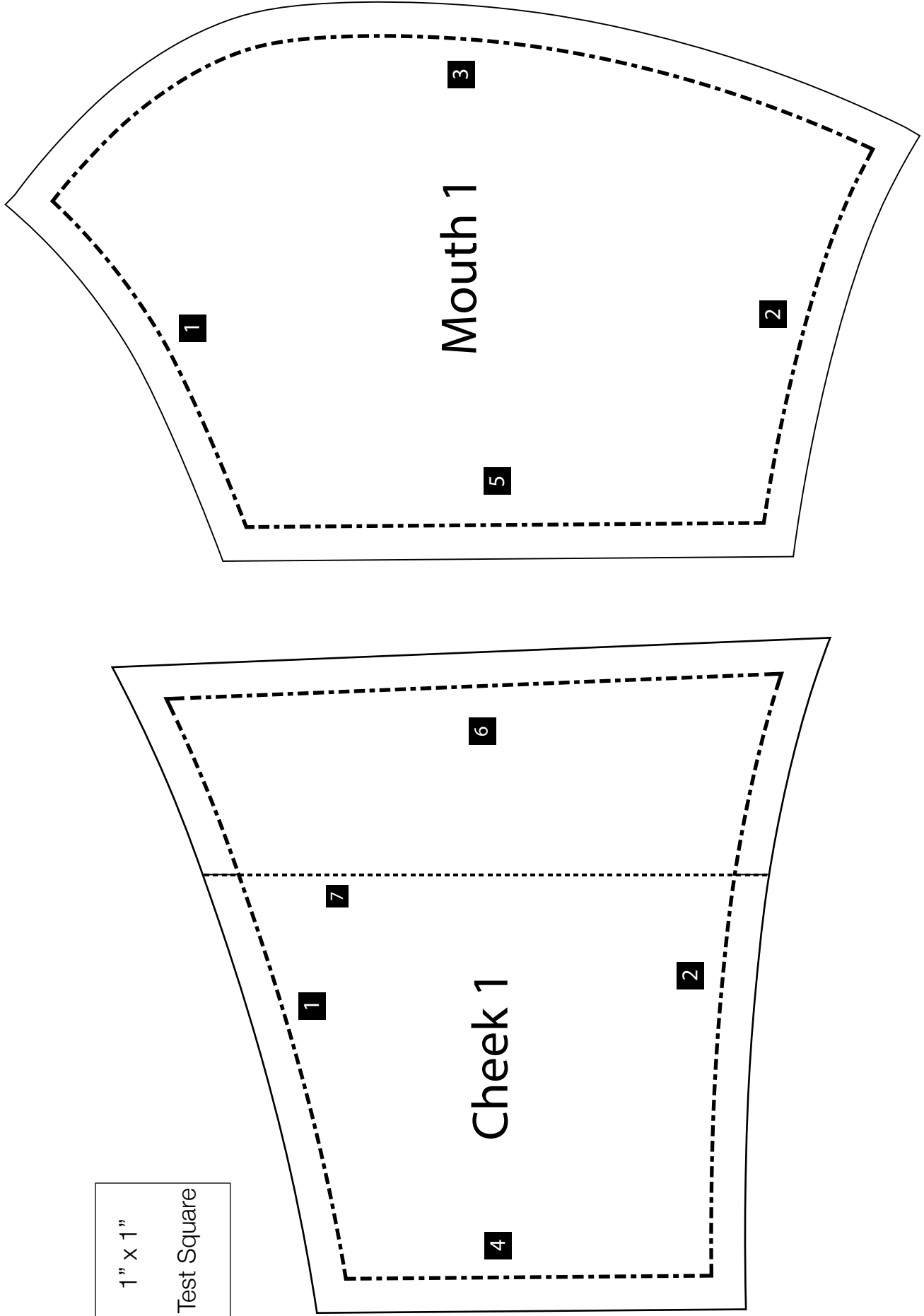

\*\*ensure that auto-fit is not selected when printing to ensure pattern measurements are not altered



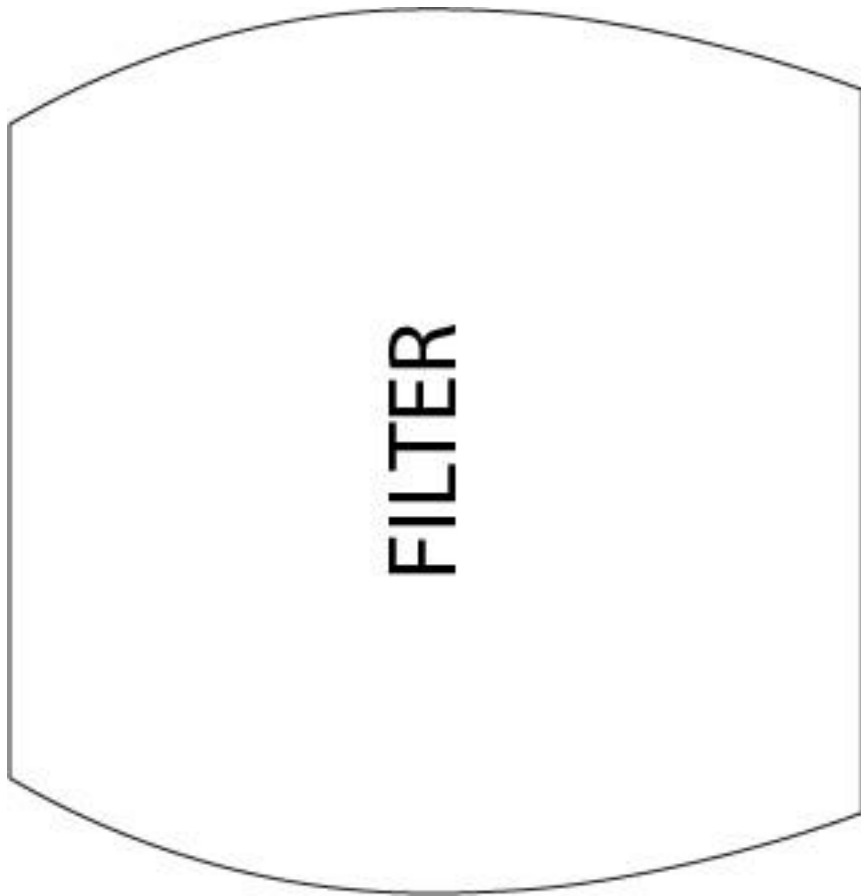

1 INCH